

EyeLink2 INSTRUCTIONS

1. Determine a yes/no response system.
2. Hold sheet with the black letters facing the patient.
3. Have the patient at the same time as you focus on the smiley face in the middle of the sheet.
4. Then, have the patient focus on the first letter of the desired word.
5. Move the sheet until your eyes “link” with the patient’s through the desired letter. It should be a straight line between you, the letter and your patient’s eyes.
6. Check with the patient to see if you are correct.
7. Continue with the process until you get the message. For ex, “ok the first letter was T. 2nd letter is E (continue to “eyelink)”.

This product is shipped with a protective film backer to protect the surface during manufacturing. Please remove this film before using the EyeLink2.

